




COMFICOMFORT® HEADBOARDS

HOW TO ASSEMBLE OR MOUNT


ALLURE / CHESTERFIELD HEADBOARDS

HEADBOARDS	ALLURE (LS) BEA830120	CHESTERFIELD (LS) BEA830100
	ALLURE (KS) BEA830130	CHESTERFIELD (KS) BEA830110

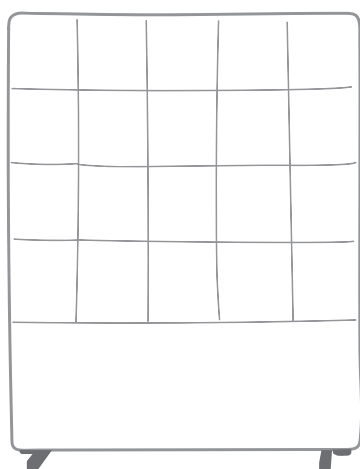

**ALLURE
HEADBOARD**


OR

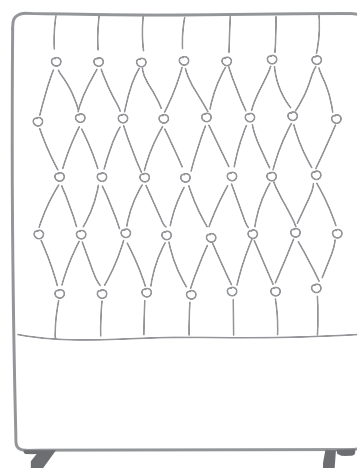

**CHESTERFIELD
HEADBOARD**

**2 X OPTIONS
TO FIT HEADBOARDS**


**OPTION 1
FREESTANDING
(WITH STABILISERS)**

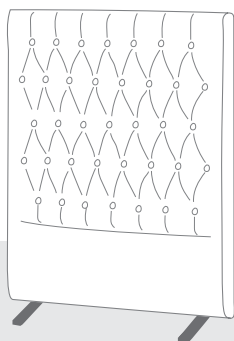

For use on the:

- ✓ RELAX BED
- ✓ CARE BED
- ✓ FIXED COMPANION BED

**OPTION 2
MOUNTED TO BED BASE
GSP640115**

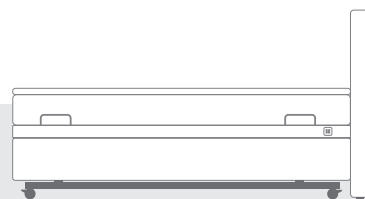


For use on the:

- ✓ RELAX BED
- ✗ CARE BED
- ✗ FIXED COMPANION BED

OPTION 1

HOW TO ASSEMBLE - FREESTANDING WITH STABILISERS

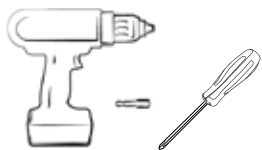


Tools Required:

- Impact driver or drill (5mm attachment)
- Phillips head screw driver (5mm)


For use on the:

- ✓ RELAX BED
- ✓ CARE BED
- ✓ FIXED COMPANION BED

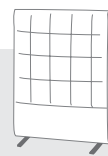


1 Flip headboard upside-down to show the black upholstery.

2 Using the leg stabiliser, align the holes to the white markings. **NOTE:** The short end of the stabiliser should face the back of the headboard (the side that will be positioned against the wall).

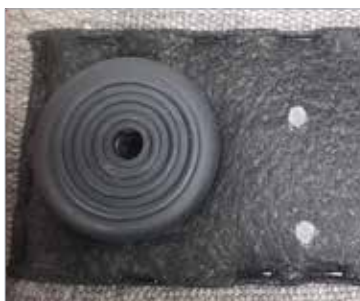

Top View

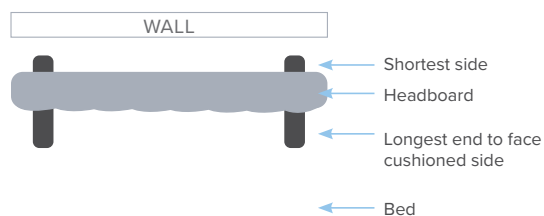

3 Secure the stabilisers in place using the screws provided.




4 Finally, flip the headboard back to its original position, enabling it to stand freely.



OPTION 2

HOW TO MOUNT TO BED

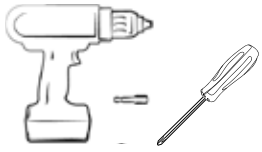

Tools Required:

- Impact driver or drill (5mm attachment)
- Phillips head screw driver (5mm)

For use on the:

✓ RELAX BED

✗ CARE BED

✗ FIXED COMPANION BED



NOTE: It is suggested that an impact driver or drill is used to speed up this process. Using a Phillips head screwdriver will work but will take more time. If the impact Driver/Drill has a clutch setting, it should be set to low torque to minimise any risk of bolt misalignment or mount point damage.

NOTE: Headboards can be mounted to the **Relax Bed only**.

CAUTION: When bed is in raised position, Headboard will raise with bed and poses risk to damaging of fixings or attachments on walls.

- 1 Raise bed height, backrest and knee break to maximum height for ease of access.

LUXE Handset



- 2 Locate mounting holes for screw insertion. To locate holes:

- a. Locate threaded mounting holes (2 per side) at head end of bed.
- b. Line up bracket over threaded mounting holes and mount in place by rotating phillips head bolt through the top and bottom mounting holes.
- c. Repeat on both sides, ensuring 4 x bolts (2 per side) are fastened in place.



- 3 When mounting brackets to bed end:
- a. Line bracket up with foot stopper (on base of headboard).
 - i. Ensure bracket is spaced approx. 40mm from edge of headboard.
 - ii. Ensure bracket is spaced approx. 60mm from base of headboard.
 - b. Through rotation the screw will pierce through the fabric and thread into wood frame (self-tap). Repeat on both sides, ensuring 8x screws (4 per side) are fastened in place.
 - c. Using an impact driver, rotate screw through the upholstery inwards towards frame.
 - d. **NOTE:** Position of bracket should be as per measurements otherwise it will not fit securely to the bed frame.



- 4 Align bracket fixings (headboard bracket will slide into bed frame bracket), Slide headboard down until both brackets click into place, the headboard is now in place.

NOTE: Headboard foot supports will rest on the ground when in lowest height.

