




STAR MOVER

The Aspire STAR Mover offers a safe and comfortable solution for patients and carers doing sit to stand transfers. Its compact and versatile design is perfect for Homecare, Institutional, and Aged Care environments.

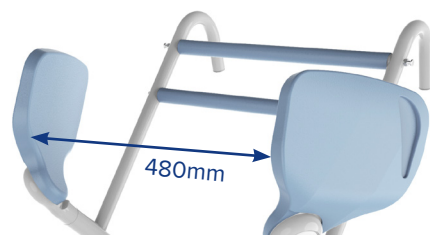


Wide Seat Entry Space



Low Footplate Height



Large Locking Castors x2

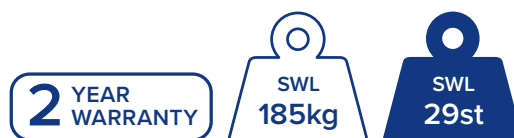

Code	Overall Length mm/"	Overall Height mm/"	Leg Length mm/"	Knee Pad Height mm/"	Seat Pad Height mm/"	Handle Height (Carer & User) mm/"	Leg Width Range mm/"	Footplate Height From Floor mm/"	Turning Circle mm/"
LSS390580	920	1030	780	330 - 380	830	1030 & 865 / 960	530 - 920	70	1060
	36	40	31	13 - 15	33	40.6 & 34 / 38	22 - 36	3	42

Note: Range of motion dimensions are accurate to a +/-10mm tolerance

Australia
1300 133 120 | aspirecare.com.au

United Kingdom
01622 541235 | aidacare.co.uk

Wide Seat Entry suits a large range of body shapes

Low Footplate Height makes patient entry and exit easier

Large 100mm Castors (two rear locking) have low rolling resistance and for smooth ride during transportation

Soft-touch Grab Bar provides a comfortable patient support for improved stability and confidence during transfers

Seat Pad Handles provide reach-around access to carers

Height-adjustable Knee Support ensures correct positioning for a wide range of patient heights

Narrow Base improves access in tight spaces

Width-adjustable Legs with tapered close position allow access between or around chair legs

HOW TO USE



1. Open legs if required and lift seat pads. Ask the user to place feet on footplate and apply brakes.



2. Ask the user to hold the grab handle and stand up.





3. Swing the seat pads back into place and ask the user to sit down. Unlock brakes and commence the transfer.

SLING COMPATIBILITY

The STAR Mover features clip attachment points that work with Aspire Deluxe Standing Slings.

Code	Description	Girth	Edging	SWL
LSS390920	Deluxe Padded Standing Sling – Polyester – Clip – Small	65 – 95cm	●	200kg
LSS390925	Deluxe Padded Standing Sling – Polyester – Clip – Medium	77 – 110cm	●	200kg
LSS390930	Deluxe Padded Standing Sling – Polyester – Clip – Large	98 – 135cm	●	300kg
LSS390935	Deluxe Padded Standing Sling – Polyester – Clip – Extra Large	118 – 157cm	●	300kg

For men and children, girth is the chest circumference | For women, girth is the hip circumference.

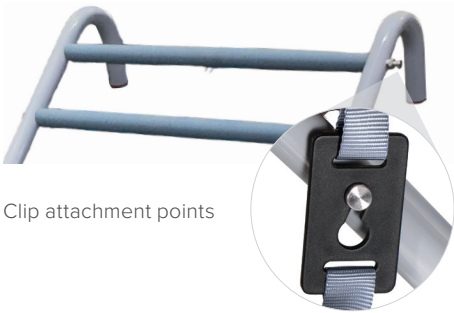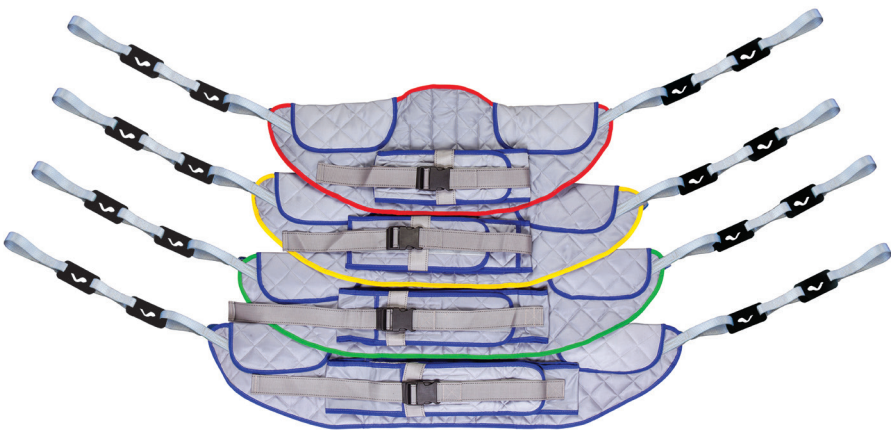

Tested to International Standards
STAR Patient Mover is registered in the ARTG, AUST R ARTG 373695